

EASY BAR REF 300.00

1. USAGE

These preformed bars facilitate the realization of a reinforced structure for all kind of fixed dental prosthesis

2. FEATURES

- Made in stainless steel
 - Different sizes considering centre to centre distance: from 6 to 30 mm
 - Thickness: 1.5 mm
 - High resistance
-

3. ADVANTAGES

- Thanks to their different sizes the Fast Protec Easy Bars could be adapted to the different conformations of the mouth and positions of the implants
 - They could be fixed with a little quantity of material
 - They could be used directly in the oral cavity to create the splintings for the dental impression.
 - They make up a stable and precise structure
 - Absence of deformation
 - High quality standard
-

4. PROCESSING

- Once poured the plaster model and put it on the articulator, realize the pink gingiva.
 - Screw the abutments on the fixtures and caliper the centre to centre distance
 - Choose the corresponding bar and put it on the abutments
 - Create the support structure
 - Control the correct match with the teeth previously assembled
 - Fasten the support structure and the teeth through the use of resin
-

5. PACKAGING

- 1 little suitcase
 - 245 bars in stainless steel from 6 to 30 mm (5 per size)
 - 1 caliper
 - 1 tweezer
 - 1 sheet of clear plastic representing the different centre to centre distances of the bars
-

6. IMPORTANT

For professional use only

7. IMPORTANT

This information is based on our present knowledge. However, this shall not constitute a guarantee for any specific product features and shall not establish a legally valid contractual relationship. The consumer is solely responsible for product appliance, if it should differ from the stated range. As it is possible that the product is used in multi-purpose use far beyond our knowledge, we do not assume liability and responsibility for other special applications than those mentioned in our MSDS or instructions for use. Despite of the terms of this MSDS, consumers have to assure themselves that the characterized product is suited for their individual intended purpose and that the safety appliance and the advices concerning environmental protection apply to their individual purpose and in their situation.

- Department issuing MSDS: P.P.M. srl di Giacomini Manuele e Monica
- Further information obtainable from: Mr. Giacomini Manuele

Please check the patient does not suffer from a known hypersensitivity towards one of the following chemical components of AISI type 304:

- C
- Mn
- Si
- P
- S
- Cr
- Ni
- Mo
- N
- Ti
- Cu